


ProteClean quickly absorbs and deodorizes sweat and dampness left on the inside of the protective clothing after being worn, thanks to the moisture-absorbing power of silica gel. This product was designed and developed to make it as easy and comfortable as possible for busy medical professionals to always wear X-ray protective clothing with comfort and security that can simply be kept on a hanger.

Structure

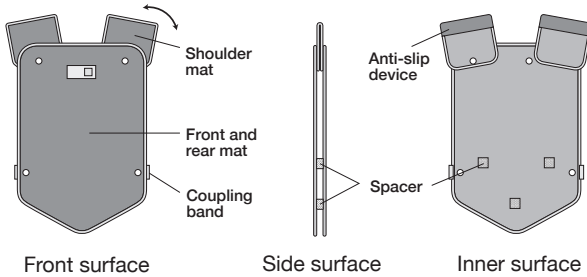

● Shoulder section is adjustable



*Sample image of the standard hanger

How to use



Fig. (1)

* Hanger not included.

Fig. (2)

Fig. (1) Hang ProteClean on the hanger

Fig. (2) Lay X-ray protective clothing over the ProteClean. Reposition the protective clothing to ensure it fully fits

Fig. (3) Moisture absorption sensor

When the blue-colored moisture-absorption sensor turns pink, please allow it to dry in a well ventilated place.



* Sample image shown is for reference only.

For Effective Use:

- The sensor may absorb moisture while in transit and turn pink. To ensure the product remains in perfect working order, please leave it on a hanger to dry in the sun or otherwise dry it out before use. (It still has a moisture-absorbing effect even if used without doing anything first)
- When the sensor turns pink in color, remove the X-ray protective clothing and allow the product to dry until the sensor changes to blue.
- Place the X-ray protective clothing as close to the product as possible. It absorbs moisture effectively.

Important Points Regarding Use

- This product is not a medical device.
- Sterilization using EOG or high-pressure steam is not possible.
- Never use in a microwave oven.
- The product is not washable. If soiled, wipe with a damp cloth.
- Do not tear open the mats containing silica gel, but dispose of them as unburnable trash.

Manufacturer **Technad** Technad Co., Ltd.
Original Seller [contact details]

MediCore Medcore Co., Ltd.

1-6-12 Hibarigaoka, Umi-machi,
Kasuya-gun, Fukuoka
TEL.+81-50-3708-1721 FAX.+81-92-510-1934
<http://www.medi-core.co.jp>

[Specifications]

Registered product name:
MediCore ProteClean
Size: 60x37x4cm (when hung)
Mat: 0.7cm thick
Cover fabric: 100% polyester
Contains: 100% silica gel
Made in Japan

